

### Intended Use

Intended for use as an uric acid standard for standardization or calibration of uricase / trinder uric acid procedures.

### Summary and Explanation

The uric acid standard is prepared from uric acid in an aqueous solution. The standard provides a concentration level of uric acid within the linearity range of the uricase / trinder procedure. Stabilizers and preservatives added.

### Principle

The standard should be used to calibrate the determination of unknown specimens using a uricase / trinder uric acid assay. Controls should be included to validate the calibration of the assay.

### Ingredients

Reactive ingredients: Uric acid solution prepared from uric acid (Free Acid). Preservatives added. Consult vial label for assigned concentration for the standard.

Precautions: For *in vitro* diagnostic use. Handle with the same precautions used for human samples.

### Storage and stability

Store at 2-8°C. Stable until expiration date if sealed tightly and stored at 2-8°C.

### Preparation

Standard is provided as a ready to use liquid.

### Procedure

The standard should be used to standardize instrument or assay following the directions that accompany the instrument and reagent kit used.

Materials provided: Aqueous Uric acid standard 5.0 mg/dl.









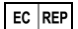
Materials not provided: Pointe Scientific uricase / trinder uric acid reagent and controls.

Limitations: Among factors that might cause inaccurate results are improper pipetting, inadequate mixing.

### Standard Value

The value is 5.0 mg/dl when used with the Pointe Scientific uricase / trinder uric acid procedure.

### Symbol Key

 Use by (YYYY-MM)	 Lot and batch code
 Catalog number	 Manufacturer
 <i>In vitro</i> diagnostic medical device	 Temperature limitation
 Consult instructions for use	 CE mark
 Authorized representative in the European Community	



Manufactured by Pointe Scientific, Inc.  
5449 Research Drive, Canton, MI 48188



Obelis s.a.  
Boulevard Général Wahis 53  
1030 Brussels, BELGIUM  
Tel: (32)2.732.59.54, Fax:(32)2.732.60.03  
email: mail@obelis.net

Rev. 5/15

P803-U7580-05