

### Intended Use

Intended for use as an albumin standard for standardization or calibration of bromocresol green (BCG) dye-binding albumin procedures.

### Summary and Explanation

The albumin standard is prepared from bovine albumin in an aqueous solution. The standard provides a level of albumin concentration within the linearity range of the bromocresol green (BCG) dye-binding procedure. Stabilizers and preservatives added.

### Principle

The standard should be used to calibrate the determination of unknown specimens using a bromocresol green (BCG) dye-binding albumin assay. Controls should be included to validate the calibration of the assay.

### Ingredients

Reactive ingredients: Albumin solution prepared from bovine albumin. Preservatives added. Consult vial label for assigned concentration for the standard.

Precautions: For *in vitro* diagnostic use. Handle with the same precautions used for human samples.

### Storage and stability

Store at 2-8°C. Stable until expiration date if sealed tightly and stored at 2-8°C.

### Preparation

Standard is provided as a ready to use liquid.

### Procedure

The standard should be used to standardize instrument or assay following the directions that accompany the instrument and reagent kit used.

Materials provided: Aqueous albumin standard 4.0 g/dl.







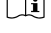
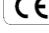
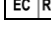
Materials not provided: Pointe Scientific Bromocresol green (BCG) dye-binding albumin reagent and controls.

Limitations: Factors that might cause inaccurate results are improper pipetting and inadequate mixing.

### Standard Value

The value is 4.0 g/dl when used with the Pointe Scientific Bromocresol green (BCG) dye-binding albumin procedure.

### Symbol Key

 Use by (YYYY-MM)	 Lot and batch code
 Catalog number	 Manufacturer
 <i>In vitro</i> diagnostic medical device	 Temperature limitation
 Consult instructions for use	 CE mark
 Authorized representative in the European Community	



Manufactured by Pointe Scientific, Inc.  
5449 Research Drive, Canton, MI 48188



Obelis s.a.  
Boulevard Général Wahis 53  
1030 Brussels, BELGIUM  
Tel: (32)2.732.59.54, Fax:(32)2.732.60.03  
email: mail@obelis.net

Rev. 1/15

P803-A7502-05